

NIST MS Search 2.0 - [MSMS Library]

File Search View Tools Options Window Help

MSMS Library

- Positive ion
 - MW=45
 - C2H7N
 - Ethylamine
 - ms2; P 46; CE 8 V
 - ms2; P 46; CE 10 V
 - ms2; P 46; CE 12 V
 - Dimethylamine
 - ms2; P 46; CE 16V
 - ms2; P 46; CE 20V
 - ms2; P 46; CE 24V
 - ms2; P 46; CE 30V
 - MW=48
 - CH4S
 - Methanethiol
 - ms2; P 48; CE 7.3 eV
 - MW=59
 - C3H9N
 - Isopropylamine
 - ms2; P 60; CE 6 V
 - ms2; P 60; CE 8 V
 - ms2; P 60; CE 10 V
 - ms2; P 60; CE 12 V
 - MW=61
 - C2H7NO
 - Ethanolamine
 - ms2; P 62.114; CE 35%
 - ms2; P 62.11; CE 35%
 - MW=62
 - MW=68
 - C3H4N2
 - Imidazole
 - ms2; P 68.9; CE 18V
 - ms2; P 68.9; CE 30V
 - ms2; P 68.9; CE 40V
 - MW=75
 - MW=82
 - MW=83
 - MW=85
 - MW=87
 - MW=88
 - MW=89
 - C3H7NO2
 - C4H11NO
 - MW=90

m/z	Relative Intensity
69.00	999.00
42.00	42.00
396.00	396.00
67.00	67.00
48.00	48.00
66.00	66.00
43.00	43.00
65.00	65.00
10.00	10.00
63.00	63.00

10 largest peaks:
69.00 999.00 | 42.00 396.00 | 67.00 48.00 | 66.00 43.00 | 65.00 10.00 |
63.00 9.00 | 71.00 9.00 | 64.00 8.00 | 40.00 4.00 | 28.00 2.00 |

Synonyms:
no synonyms.


(nist_msms) Imidazole

Name: Imidazole
Formula: C₃H₄N₂
Other DBs: None
Comment: NIST Mass Spectrometry Data Center
Instrument type: QqQ/triple quadrupole
Spectrum type: ms2
Compound type: M
Precursor type: [M+H]⁺
Precursor m/z: 68.9
Collision energy: 18V
Instrument: Micromass quattro
Sample inlet: direct flow injection
Ionization: ESI
Ion mode: P
Collision gas: Ar
Cone voltage: 30V
AUX: Condition: S=MW(1%, H+) C.=5, T=120/150, U=700,p=1.6 Description: mmsms_2_40-30-p69(100) Origin: Sigma-Aldrich Catalog: 10250

Plot/Text / Plot

Lib. Search Other Search Names Compare Librarian MSMS

For Help, press F1

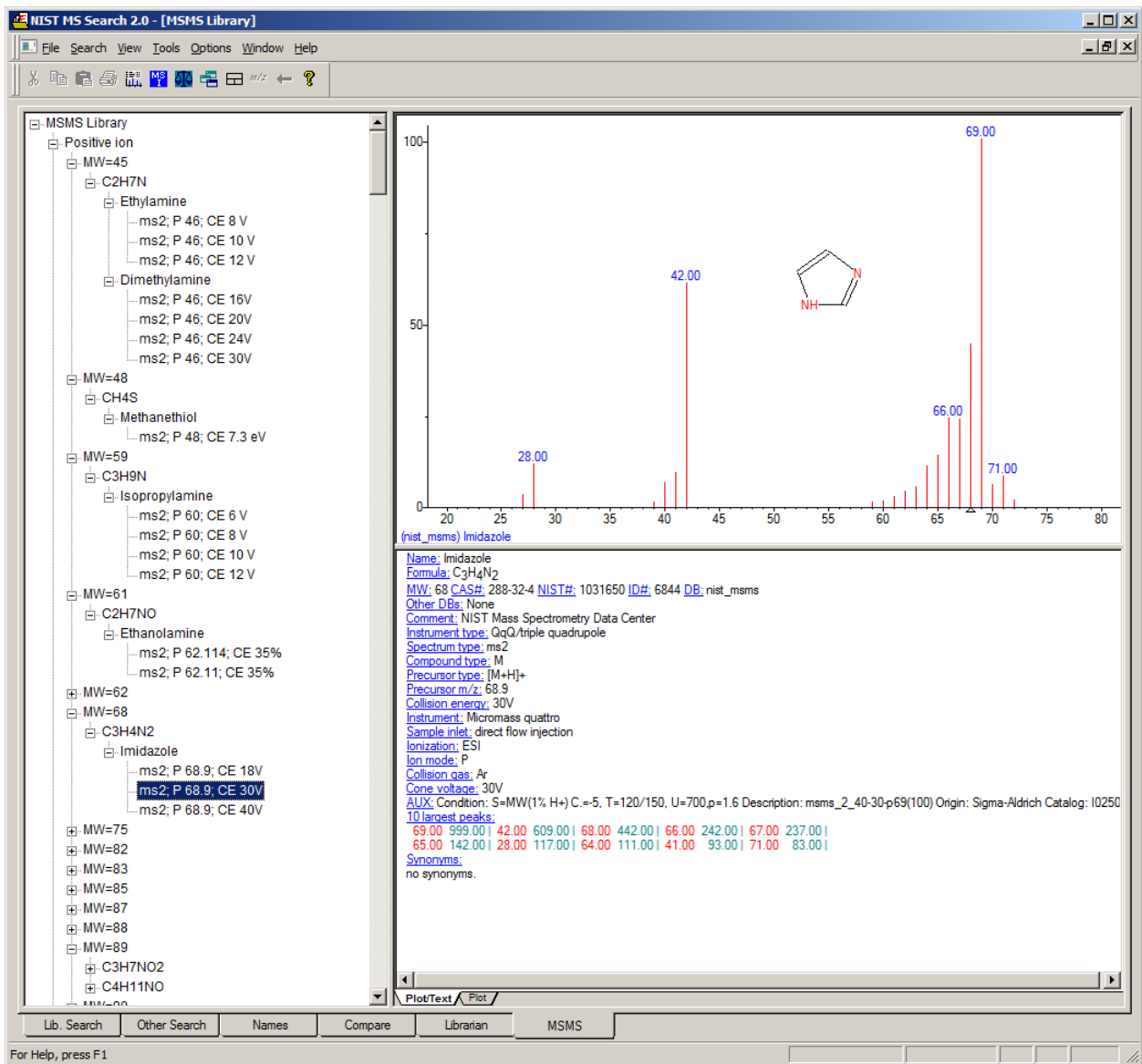

NIST MS Search 2.0 - [MSMS Library]

File Search View Tools Options Window Help

MSMS Library

- Positive ion
 - MW=45
 - C2H7N
 - Ethylamine
 - ms2; P 46; CE 8 V
 - ms2; P 46; CE 10 V
 - ms2; P 46; CE 12 V
 - Dimethylamine
 - ms2; P 46; CE 16V
 - ms2; P 46; CE 20V
 - ms2; P 46; CE 24V
 - ms2; P 46; CE 30V
 - MW=48
 - CH4S
 - Methanethiol
 - ms2; P 48; CE 7.3 eV
 - MW=59
 - C3H9N
 - Isopropylamine
 - ms2; P 60; CE 6 V
 - ms2; P 60; CE 8 V
 - ms2; P 60; CE 10 V
 - ms2; P 60; CE 12 V
 - MW=61
 - C2H7NO
 - Ethanolamine
 - ms2; P 62.114; CE 35%
 - ms2; P 62.11; CE 35%
 - MW=62
 - MW=68
 - C3H4N2
 - Imidazole
 - ms2; P 68.9; CE 18V
 - ms2; P 68.9; CE 30V
 - ms2; P 68.9; CE 40V**
 - MW=75
 - MW=82
 - MW=83
 - MW=85
 - MW=87
 - MW=88
 - MW=89
 - C3H7NO2
 - C4H11NO

100
50
0

(nist_msms) Imidazole

Name: Imidazole
 Formula: C₃H₄N₂
 MW: 68 CAS#: 288-32-4 NIST#: 1031655 ID#: 6845 DB: nist_msms
 Other DBs: None
 Comment: NIST Mass Spectrometry Data Center
 Instrument type: QqQ/triple quadrupole
 Spectrum type: ms2
 Compound type: M
 Precursor type: [M+H]⁺
 Precursor m/z: 68.9
 Collision energy: 40V
 Instrument: Micromass quattro
 Sample inlet: direct flow injection
 Ionization: ESI
 Ion mode: P
 Collision gas: Ar
 Cone voltage: 30V
 AUX: Condition: S=MW(1%; H+) C.=5, T=120/150, U=700,p=1.6 Description: msms_2_40-30-p69(100) Origin: Sigma-Aldrich Catalog: 10250

10 largest peaks:
 69.00 999.00 | 68.00 783.00 | 67.00 492.00 | 42.00 443.00 | 70.00 367.00 |
 66.00 320.00 | 28.00 311.00 | 41.00 296.00 | 64.00 193.00 | 63.00 177.00 |

Synonyms:
 no synonyms.

Plot/Text/Plot

Lib. Search Other Search Names Compare Librarian MSMS

For Help, press F1

NIST MS Search 2.0 - [MSMS Library]

File Search View Tools Options Window Help

MSMS Library

- Positive ion
- Negative ion
 - MW=76
 - C2H4O3
 - Glycolic acid
 - ms2; P 151.004; CE 35%
 - ms2; P 151.00; CE 35%
 - C3H8O2
 - 1,2-Propanediol
 - ms2; P 75.191; CE 35%
 - ms2; P 75.21; CE 35%
 - MW=85
 - NNaO3
 - Sodium nitrate
 - ms2; P 147; CE 28 %
 - ms2; P 402; CE 19 %
 - ms2; P 487; CE 18 %
 - MW=86
 - C4H6O2
 - 3-Butenoic acid
 - ms2; P 85.195; CE 35%
 - ms2; P 85.24; CE 35%
 - C5H10O
 - cis-2-Penten-1-ol
 - ms2; P 103.288; CE 35%
 - ms2; P 103.29; CE 35%**
 - ms2; P 102.156; CE 35%
 - ms2; P 102.23; CE 35%
 - MW=88
 - MW=90
 - MW=91
 - MW=95
 - MW=96
 - MW=98
 - MW=99
 - MW=100
 - MW=101
 - MW=102
 - MW=103
 - MW=104
 - MW=106
 - MW=109
 - MW=110
 - MW=111

Plot of Text / Plot

(nist_msms) cis-2-Penten-1-ol

Name: cis-2-Penten-1-ol
 Formula: C₅H₁₀O
 MW: 96 CAS#: 1576-95-0 NIST#: 1034031 ID#: 8370 DB: nist_msms
 Other DBs: None
 Comment: NIST Mass Spectrometry Data Center
 Instrument type: IT/ion trap
 Spectrum type: ms2
 Compound type: M
 Precursor type: [M-H+H2O]
 Precursor m/z: 103.29
 Collision energy: resonant relative/normalized 35%
 Instrument: Thermo Finnigan LTQ
 Ionization: ESI
 Ion mode: N
 Pressure: 0.009mTorr
 AUX: Consensus spectrum; 0.1mg/mL in 50% MeOH/0.1% TFA
 4 largest peaks:
 59.10 999.00 | 57.10 18.00 | 43.20 7.00 | 42.00 3.00 |
 Synonyms:
 no synonyms.

Lib. Search Other Search Names Compare Librarian MSMS

For Help, press F1

NIST MS Search 2.0 - [MSMS Library]

File Search View Tools Options Window Help

MSMS Library

- Positive ion
- Negative ion
 - MW=76
 - C2H4O3
 - Glycolic acid
 - ms2; P 151.004; CE 35%
 - ms2; P 151.00; CE 35%
 - C3H8O2
 - 1,2-Propanediol
 - ms2; P 75.191; CE 35%
 - ms2; P 75.21; CE 35%
 - MW=85
 - NNaO3
 - Sodium nitrate
 - ms2; P 147; CE 28 %
 - ms2; P 402; CE 19 %
 - ms2; P 487; CE 18 %
 - MW=86
 - C4H6O2
 - 3-Butenoic acid
 - ms2; P 85.195; CE 35%
 - ms2; P 85.24; CE 35%
 - C5H10O
 - cis-2-Penten-1-ol
 - ms2; P 103.288; CE 35%
 - ms2; P 103.29; CE 35%
 - ms2; P 102.156; CE 35%**
 - ms2; P 102.23; CE 35%
 - MW=88
 - MW=90
 - MW=91
 - MW=95
 - MW=96
 - MW=98
 - MW=99
 - MW=100
 - MW=101
 - MW=102
 - MW=103
 - MW=104
 - MW=106
 - MW=109
 - MW=110
 - MW=111

Plot of Text / Plot

(nist_msms) cis-2-Penten-1-ol

Name: cis-2-Penten-1-ol
 Formula: C₅H₁₀O
 MW: 96 CAS#: 1576-95-0 NIST#: 1034032 ID#: 8371 DB: nist_msms
 Other DBs: None
 Comment: NIST Mass Spectrometry Data Center
 Instrument type: IT/ion trap
 Spectrum type: ms2
 Compound type: M
 Precursor type: [M-H+NH3]
 Precursor m/z: 102.156
 Collision energy: resonant relative/normalized 35%
 Instrument: Thermo Finnigan LTQ
 Ionization: ESI
 Ion mode: N
 Pressure: 0.009mTorr
 AUX: 0.1mg/mL in 50% MeOH/0.2% ammonium hydroxide
 6 largest peaks:
 58.10 999.00 | 42.12 260.00 | 56.14 19.00 | 33.19 16.00 | 85.00 12.00 |
 60.01 8.00 |
 Synonyms:
 no synonyms.

Lib. Search Other Search Names Compare Librarian MSMS

For Help, press F1

About NIST MS Search 2.0



The NIST Mass Spectral Search Program
for the NIST/EPA/NIH Mass Spectral Library
Version 2.0f, build Oct 8 2008

Software by S. Stein, Y. Mirokhin, D. Tchekhovskoi, and G. Mallard.
Data Evaluation by A. Mikaia, V. Zaikin, J. Little, Damo Zhu, E. White and D.
Sparkman.

NIST uses its best effort to deliver a high quality copy of the database and to ensure that the data shown are accurate. However, NIST makes no warranties to that effect, and NIST shall not be liable for any damage that may result from errors or omissions in the database.

Distributed by the Standard Reference Data Program of the National Institute of Standards and Technology. Copyright © 2008 by the U.S. Secretary of Commerce on behalf of the United States of America. All Rights Reserved.

Portions copyright © 1984-1996 FairCom Corporation. "FairCom" and "c-tree Plus" are trademarks of Faircom Corporation and are registered in the United States and other countries. All Rights Reserved.

Serial Number
Not Applicable
Distributed

Scientific Instrument Services, Inc.
1027 Old York Rd., Ringoes, NJ 08551
PHONE: (908)788-5550; FAX: (908)806-6631
<http://www.sisweb.com/nist>

OK